

Error Codes

This device includes error codes that indicate problems with the unit. When an error occurs, the device displays the letters “Er” and a two-digit code (table 2).

Table 2: Error Codes

Error Code	Description
01	Configuration sector error
02	Patient data pointer error
03	Main memory pointer error (Device memory is intact; however, the most recent session may be missing from the device.)
04	Data format 13 stored packet pointer error
05	Main data format 13 pointer error (Device memory is intact; however, the most recent stored measurement may be missing from the device.)

Some error codes may be corrected by the user. See “Troubleshooting” for more information.