



Model 7500 Quick Start-up Guide

SET USER-DEFINED DEFAULTS

SET PATIENT SECURITY MODE ON

DATA COLLECTION & DOWNLOAD

TO CLEAR MEMORY

1. Turn unit on – Review factory settings below
2. Step through ALL settings using LIMITS button
3. Change desired settings using or buttons
4. MUST push LIMITS button after each setting to confirm & move on
5. Scroll to end of settings to 2nd symbol (=minutes) and press LIMITS button
6. Unit will automatically review new settings
7. TO SAVE SETTINGS – press and hold ALARM SILENCE button while briefly pressing LIMITS button – “dEF On” will flash

Return to Factory Defaults

Note: User-Defined Default values are lost when returned to Factory Defaults.

1. Turn unit on – “dEF On” will appear
2. Press and hold ALARM SILENCE button and briefly press button – “dEF OFF” will flash
3. Press POWER button until unit turns OFF, then turn back ON

Factory Default Settings

The values at right are pre-programmed in the Model 7500. The Patient Security Mode is OFF, unless turned on.

KEY	Parameter	Factory Setting	Adjustment Options	Increment
	SpO ₂ Low Alarm Limit	85%	Off, 50-95	1%
	Pulse Rate High Alarm Limit	200 BPM	Off, 75-275	5 BPM
	Pulse Rate Low Alarm Limit	50 BPM	Off, 30-110	5 BPM
	SpO ₂ High Alarm Limit	Off	Off, 80-100	1%
	Alarm Volume	High	Off, Low, High	N/A
	Oximetry Averaging	No	Yes, No	N/A

1. Turn unit on – Review factory settings below
2. Step through ALL settings using LIMITS button
3. Change desired settings using or buttons
4. MUST push LIMITS button after each setting
5. Scroll to end of settings to 2nd symbol (=minutes) and press LIMITS button
6. Unit will automatically review new settings
7. TO SAVE SETTINGS – press and hold ALARM SILENCE button while briefly pressing LIMITS button – “dEF On” will flash
8. Press POWER button until unit turns OFF
9. TO SET – Press and hold ALARM SILENCE button while turning the unit back ON
10. Screen will flash “SEC On”

Note: To set “Low O₂” alarm below 85%, must set in User-Defined Defaults first, then follow steps above.

Turn Off Patient Security Mode

1. Press POWER button until unit turns OFF
2. Together hold down ALARM SILENCE button, then LIMITS button and then POWER button
3. Units reads “SEC OFF”

Data Collection

1. Turn unit on – Recording begins automatically
- Note:** Data less than 1 minute is not saved.


Downloading Data To PC

Note: Patient Security Mode must be OFF

1. Press POWER button until unit turns OFF
2. Connect unit to PC using 7500SC download cable
3. Boot up nVISION and select FILE>New Data Capture, click 7500 oximeter option
4. On 7500 unit press and hold and POWER buttons - unit will display “PLA bAC” during download
5. Unit returns to normal operating mode after download

1. Turn unit on
- Note:** Patient Security Mode must be OFF
2. Press LIMITS button until “CLr no” is displayed
 3. Press button until “CLr YES” appears
 4. Press LIMITS button to confirm
 5. “dEL no” appears, press button so “dEL YES” appears
 6. Press LIMITS button to confirm
 7. “dnE CLr” confirms memory has been cleared

For complete specifications, please refer to the Operator’s Manual.

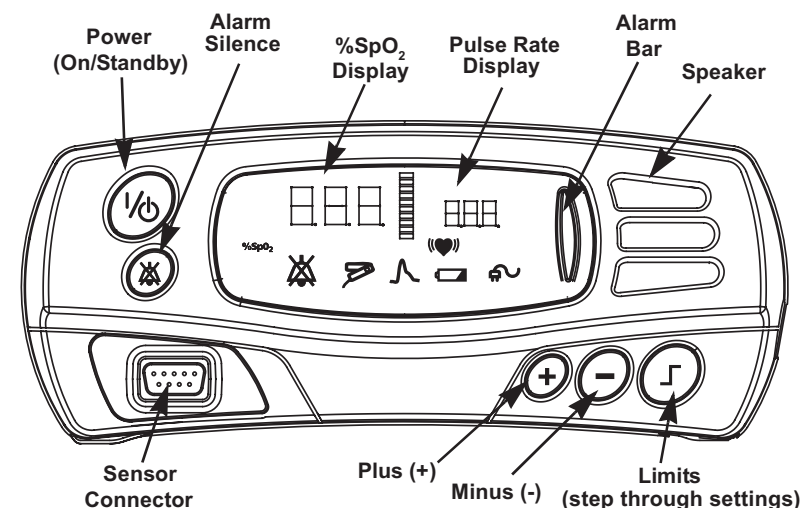



<https://www.nonin.com/support/7500/>

EN: Translations of the Model 7500 Operator's Manual and Quick Start Guide can be found using this QR code.

FR: Pour accéder aux traductions du Manuel d'utilisation et du Guide de mise en marche rapide, veuillez scanner ce code QR.

DE: Übersetzungen des Benutzerhandbuchs für Modell 7500 und der Schnellstartanleitung können mithilfe dieses QR-Codes aufgerufen werden.

IT: Le traduzioni del Manuale dell'operatore e della Guida rapida del modello 7500 sono disponibili utilizzando questo codice QR.

ES: Puede encontrar traducciones del Manual del operador y de la Guía de inicio rápido del modelo 7500 utilizando este código QR.

PT: Utilize o seguinte código QR para encontrar traduções do Manual do Operador do Modelo 7500.

NL: Vertalingen van de gebruikershandleiding van model 7500 zijn te vinden door deze QR-code te scannen.

EL: Μπορείτε να βρείτε μεταφράσεις του Εγχειριδίου χρήσης του μοντέλου 7500 χρησιμοποιώντας αυτόν τον κωδικό QR.

DA: Oversættelser af brugermanualen til Model 7500 kan findes ved hjælp af denne QR-kode.

SV: Översättningar av användarhandboken för Model 7500 kan hittas med denna QR-kod.

FI: Voit hakea mallin 7500 käyttöoppaan käännettyjä versioita täällä QR-koodilla.

PO: Przekłady podręcznika operatora modelu 7500 można znaleźć, używając tego kodu QR.

NO: Oversettelser av brukerhåndboken for modell 7500 kan bli funnet ved å bruke denne QR-koden.

Model 7500

Quick Start-up Guide

