Instructions for Use—English

Model 6000CP/7000P Pediatric Disposable Single-Patient Use Pulse Oximeter Sensor

Cautions: Federal law (USA) restricts this device to sale by or on the order of a licensed practitioner.

Indications for Use: Nonin Models 6000CP and 7000P Single-Patient Use Disposable Pulse Oximeter Sensors are indicated for non-invasive spot-checking and/or continuous monitoring of pediatric patients who are well or poorly perfused, weighing greater than 22 pounds (10 kilograms).

Contraindications: Do not use the device in an MR environment or in an explosive atmosphere. This device is not defibrillation proof per IEC 60601-1:1990 clause 17h.

Warnings: The use of sensor and oximeter combinations other than Nonin-branded products have not been tested for accuracy as a system and may affect performance of the system. Refer to Nonin pulse oximeter operator's manuals for a complete listing of Nonin-branded oximeters, sensors, and accessories.

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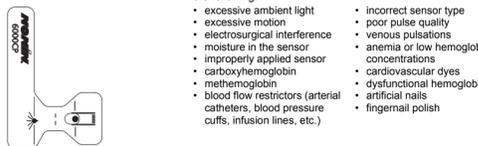
Table with 2 columns: Symbol and Definition of Symbol. Includes symbols for CE marking, caution, authorized representative, manufacturer, lot number, use by, and storage/shipping temperature range.

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Choosing the Sensor Application Site: The preferred application site for pediatric patients is the index finger. However, other fingers or toes may be used where the tissue thickness is between 5 and 17 millimeters (see figure at left).

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Attaching the Pediatric Disposable Sensor: 1. Partially peel away the adhesive backing from the sensor, as shown in figure A. 2. Place the patient's fingernail against the sensor. Center the tip of the digit against the center line marked on the sensor.

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