



Instructions for Use—English

Serial Interface Cable: 7500SC

Nonin’s serial interface cables are designed for connecting Nonin pulse oximeters and a personal computer (PC) when using nVISION® software. When users preset the pulse oximeter for Memory Playback/Output mode and have nVISION software initialized on their PC, data can be transferred between the pulse oximeter and the PC.

Notes:

- Disconnect the serial interface cable when it is not in use.
- The 7500SC cable is developed specifically for use with Nonin’s 7500 series pulse oximeters.
- The 7500SC cable may be used for real-time data collection to a PC; however, this is not a Nonin-supported application.

Setup:

1. Connect the cable end labeled “computer” to the serial port on your computer.
2. Connect the other cable end to the external interface port on the rear of the pulse oximeter.
3. Initialize your nVISION software, and ensure that the correct port has been selected.
4. Select “New Data Capture” from the File pull-down menu, select oximeter Model 7500 Series, and follow the instructions in the on-screen dialog box. Observe that as data is captured, it is listed in the “Captured Recordings” window.



Nonin Medical, Inc.
13700 1st Avenue North
Plymouth, MN 55441, USA
www.nonin.com

E-mail: info@nonin.com
+1 (763) 553-9968
(800) 356-8874 (US and Canada)



+31 (0)13 - 45 87 130
(Europe)MPS,
Medical Product Service GmbH

Bomgasse 20
D-35619 Braunfels, Germany



Nonin Medical B.V.
Doctor Paul Janssenweg 150
5026 RH Tilburg, Netherlands



MedEnvoy Switzerland
Gotthardstrasse 28, 6302 Zug
Switzerland

Email: infointl@nonin.com (Europe)