



Operator's Manual

Model 7500

Pulse Oximeter

R_X Only


CE 0123

English



Follow Instructions for Use.

Nonin reserves the right to make changes and improvements to this manual and the products it describes at any time, without notice or obligation.

 **Nonin Medical, Inc.**
13700 1st Avenue North
Plymouth, MN 55441, USA

+ 1 (763) 553-9968
800-356-8874 (USA and Canada)
Fax: + 1 (763) 553-7807
E-mail: info@nonin.com

Nonin Medical B.V.
Doctor Paul Janssenweg 150
5026 RH Tilburg, Netherlands

+31 (0)13 - 45 87 130 (Europe)
E-mail: infointl@nonin.com

nonin.com

CE 0123



MPS, Medical Product Service GmbH
Borngasse 20
D-35619 Braunfels, Germany

References to “Nonin” in this manual shall imply Nonin Medical, Inc.

Nonin, PureLight, and nVISION are registered trademarks or trademarks of Nonin Medical, Inc.

© 2021 Nonin Medical, Inc.
114514-001-01 07/2021

Contents

Indications for Use	1
Warnings	1
Cautions	2
Guide to Symbols	4
Displays, Indicators and Controls.....	6
%SpO ₂ Display.....	6
Pulse Rate Display	6
Numeric LEDs	6
Indicators and Icons	7
Model 7500 Front Panel Buttons	8
Operating the Model 7500	9
Operating Modes and Defaults.....	10
Setup Mode, Viewing Limits and Setting Time	10
Factory Defaults	10
User-Defined Defaults	11
Patient Security Mode	11
Viewing and Changing Patient Security Mode.....	12
Operator Functions	13
Care and Maintenance.....	16
Cleaning the Model 7500.....	16
Alarms and Limits.....	17
High Priority Alarms.....	17
Medium Priority Alarms	17
Watchdog Alarms	17
Informational Tones.....	17
Alarm Summary.....	18
Patient Alarms.....	18
Equipment Alarms.....	18
Reviewing and Setting Volume and Alarm Limits	19
Reviewing, Setting, or Changing Volumes and Alarm Limits	19
Silencing Alarms.....	19
Recalling Previous Settings.....	20
Error Codes	20
Memory and Data Output Features	21
Serial Patient Data Output.....	21
Analog Output.....	22
Analog Output Calibration.....	22
Memory Features	23

Contents (Continued)

Clearing Patient Memory	23
Playing Back Memory Data	23
Connecting the Device into a Medical System	24
Service, Support, and Warranty	25
Warranty	25
Parts and Accessories	26
Troubleshooting	27
Technical Information	29
Manufacturer's Declaration	29
Essential Performance	29
Oximetry Averaging for Displayed SpO ₂	31
Equipment Response Time	31
Testing Summary	32
SpO ₂ Accuracy Testing	32
Pulse Rate Motion Testing	32
Low Perfusion Testing	33
Principles of Operation	33
Specifications	34

Figures

Figure 1. Model 7500 Front View	6
---------------------------------------	---

Tables

Table 1. Symbols	4
Table 2. Factory Default Settings.....	10
Table 3. Basic Functions.....	13
Table 4. Limits Display Sequence.....	13
Table 5. Advanced Options.....	14
Table 6. Patient Alarms.....	18
Table 7. Equipment Alarms.....	18
Table 8. Real-time Data Output	21
Table 9. Electromagnetic Emissions.....	29
Table 10. Electromagnetic Immunity.....	30



Indications for Use

The Nonin® Model 7500 Pulse Oximeter is a portable, tabletop device indicated for use in simultaneously measuring, displaying, and recording functional oxygen saturation of arterial hemoglobin (SpO₂) and pulse rate of adult, pediatric, infant, and neonatal patients. It is intended for spot-checking and/or continuous monitoring of patients during both motion and no-motion conditions, and for patients who are well or poorly perfused.

Warnings

Do not use in an explosive atmosphere or in the presence of flammable anesthetics or gases.
Do not use this device in an MRI environment.
This device is not defibrillation proof per IEC 60601-1.
This device is intended only as an adjunct device in patient assessment. It must be used in conjunction with other methods of assessing clinical signs and symptoms.
Oximeter readings of this device may be affected by the use of an electrosurgical unit (ESU).
Inspect the sensor application site at least every 4 hours to ensure correct sensor alignment and skin integrity. Patient sensitivity to sensors and/or double-backed adhesive strips may vary due to medical status or skin condition.
Avoid excessive pressure to the sensor application site as this may cause damage to the skin beneath the sensor.
To avoid patient injury, use only Nonin-branded PureLight® pulse oximeter sensors. These sensors are manufactured to meet the accuracy specifications for Nonin pulse oximeters. Using other manufacturers' sensors can result in improper pulse oximeter performance.
To prevent improper performance and/or patient injury, verify compatibility of the monitor, sensor(s), and accessories before use.
No modifications to this device are allowed as it may affect device performance.
Do not use a damaged sensor. If the sensor is damaged in any way, discontinue use immediately and replace the sensor.
Verify all device settings, alarm settings and limits in setup mode to ensure that they are set as intended.
A hazard can exist if different presets are used on multiple 7500 monitors in one care area.
Do not use this device in or around water or any other liquid, with or without AC power.
As with all medical equipment, carefully route patient cables and connections to reduce the possibility of entanglement, strangulation, or injury to the patient.
Use this device only with Nonin-specified power supplies.
This device turns off after approximately 30 minutes when in low battery mode.
This device should not be used adjacent to or stacked with other equipment. If adjacent or stacked use is necessary, the device should be observed carefully to verify normal operation.
The battery pack must be installed at all times while the device is operating—even when operating on AC power. Do NOT use the device without batteries.
The use of accessories, sensors, and cables other than those listed in the Parts and Accessories List may result in increased electromagnetic emission and/or decreased immunity of this device.

Warnings (Continued)

To comply with relevant product safety standards, ensure that all alarm volumes are set appropriately and are audible in all situations. Do not cover or otherwise hinder any speaker openings.

Portable RF communications equipment such as cell phones or radios (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the ME system, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

Cautions

When mounting the monitor to a mobile pole, mounting the monitor more higher than 1.5 meters (5 feet) or mounting more than 2 kilograms (4.4 pounds) of equipment onto the pole may result in tipping, damage to the equipment, or injury.

This equipment complies with IEC 60601-1-2 for electromagnetic compatibility for medical electrical equipment and/or systems. This standard is designed to provide reasonable protection against harmful interference in a typical medical installation. However, because of the proliferation of radio-frequency transmitting equipment and other sources of electrical noise in healthcare and other environments, it is possible that high levels of such interference due to close proximity or strength of a source might disrupt the performance of this device. Medical electrical equipment needs special precautions regarding EMC, and all equipment must be installed and put into service according to the EMC information specified in this manual.

If this device fails to respond as described, discontinue use until the situation is corrected by Nonin Technical Service.

Review all limits to ensure they are appropriate for the patient.

Setting alarm limits to extremes can render the alarm system useless.

Do not simultaneously touch the accessible connector pins and the patient.

The sensor might not work on cold extremities due to reduced circulation. Warm or rub the finger to increase circulation, or reposition the sensor.

Do not gas sterilize or autoclave this device.

Batteries might leak or explode if used or disposed of improperly.

This device has motion tolerant software that minimizes the likelihood of motion artifact being misinterpreted as good pulse quality. In some circumstances, however, the device may still interpret motion as good pulse quality.

This device is a precision instrument and must be repaired by Nonin Technical Service. Field repair of the device is not possible. Do not attempt to open the case or repair the electronics. Opening the case may damage the device and void the warranty.

Do not place liquids on top of this device.

Do not immerse this device or sensors in any liquids.

Do not use caustic or abrasive cleaning agents on the unit or sensors.

Follow local, state and national governing ordinances and recycling instructions regarding disposal or recycling of the device and device components, including batteries. Use only Nonin-approved battery packs.

To prevent potential loss of monitoring, do not use ear clip or reflective sensors on pediatric or neonatal patients.

In compliance with the European Directive on Waste Electrical and Electronic Equipment (WEEE) 2002/96/EC, do not dispose of this product as unsorted municipal waste. This device contains WEEE materials; please contact your distributor regarding take-back or recycling of the device. If you are unsure how to reach your distributor, please call Nonin for your distributor's contact information.

To prevent potential loss of monitoring or inaccurate data, remove any objects that might hinder pulse detection and measurement (e.g., blood pressure cuffs).



Cautions (Continued)

If the entire memory is filled, portions of the oldest record will be overwritten when a new record begins.	
This device is designed to determine the percentage of arterial oxygen saturation of functional hemoglobin. Factors that may degrade pulse oximeter performance or affect the accuracy of the measurement include the following:	
<ul style="list-style-type: none"> - excessive ambient light - excessive motion - electro-surgical interference - blood flow restrictors (arterial catheters, blood pressure cuffs, infusion lines, etc.) - moisture in the sensor - improperly applied sensor - incorrect sensor type - poor pulse quality - venous pulsations 	<ul style="list-style-type: none"> - anemia or low hemoglobin concentrations - cardiogreen and other intravascular dyes - carboxyhemoglobin - methemoglobin - dysfunctional hemoglobin - artificial nails or fingernail polish - a sensor not at heart level - residue (e.g., dried blood, dirt, grease, oil) in the light path.
When using the monitor in the home, avoid exposing the monitor to lint and dust.	
When using the monitor around small children and pets, avoid leaving the monitor unattended. Cables pose a risk of injury, including strangulation.	
A functional tester cannot be used to assess the accuracy of a pulse oximeter monitor or sensor.	
All parts and accessories connected to the serial port of this device must be certified according to at least IEC Standard EN 60950, IEC 62368-1, or UL 1950 for data-processing equipment.	
Operation of this device below the minimum amplitude of 0.3% modulation may cause inaccurate results.	
Do not use the power supply if the integrity of the AC cord conductors or the outlet is in doubt.	
Failure of a network data coupling (serial cable/connectors) will result in loss of data transfer.	

Guide to Symbols

This table describes the symbols that are found on the Model 7500. Detailed information about functional symbols can be found in “Operating the Model 7500.”

Table 1: Symbols


































Symbol	Description
	Caution!
	Consult instructions for use.
	Follow instructions for use.
	Type BF Applied Part (Patient isolation from electrical shock).
	UL Mark for Canada and the United States with respect to electric shock, fire, and mechanical hazards only in accordance with: <ul style="list-style-type: none"> • ANSI/AAMI ES60601-1:2005/(R)2012 and CAN/CSA-C22.2 No. 60601-1:14 • ISO 80601-2-61:2011, IEC 60601-1-8: 2006+A1:2012
CE 0123	CE Marking indicating conformance to EC directive No. 93/42/EEC concerning medical devices.
	Serial Number
	Indicates separate collection for electrical and electronic equipment (WEEE).
	Authorized Representative in the European Community.
	Manufacturer
	Catalogue Number
	Quantity
	Date of Manufacture
	Country of Manufacture
	Storage/Shipping Temperature Range
	RoHS Compliant (China)

Table 1: Symbols (Continued)

Symbol	Description
	Federal law (USA) restricts this device to sale by or on the order of a licensed practitioner.
	Medical Device
	Lot Number
IP33	Protected against spraying water and against access to hazardous parts with a tool, per IEC 60529.
%SpO₂	%SpO ₂ Display
	Pulse Rate Display
	Numeric LEDs
	Alarm Bar LED
	Pulse Quality LED
	Sensor Alarm LED
	Pulse Strength Bargraph LED
	Alarm Silence LED
	AC Power Supply LED
	Low Battery LED
	ON/STANDBY Button
	Alarm Silence Button
	Limits Button
	Plus Button
	Minus Button
	Non-ionizing electromagnetic radiation. Equipment includes RF transmitters; interference may occur in the vicinity of equipment marked with this symbol.

Displays, Indicators and Controls

This section describes the displays, indicators, and controls for the Model 7500.

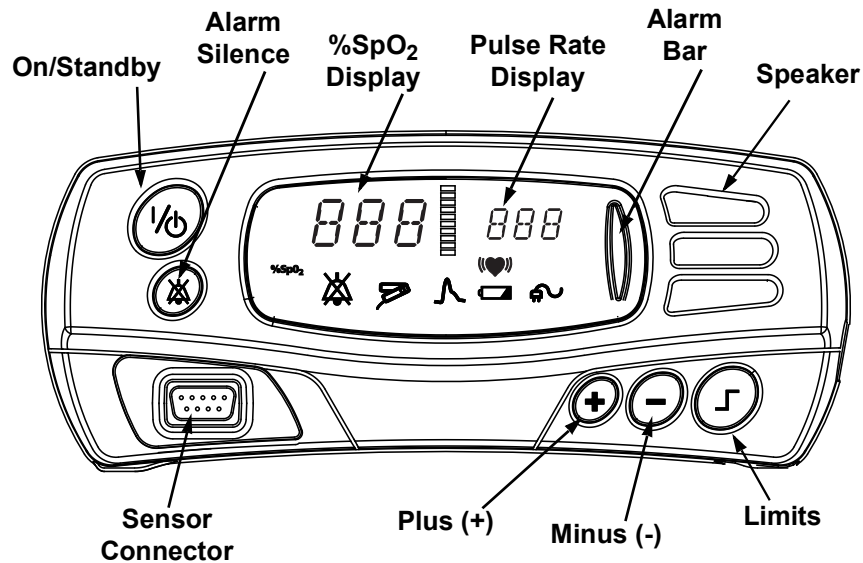


Figure 1: Model 7500 Front View

%SpO₂ Display

The %SpO₂ display is located on the left-hand side of the Model 7500 front panel and is identified by the %SpO₂ symbol. This display shows blood oxygen saturation, from 0 to 100 percent. The numeric displays blink during SpO₂ alarm conditions. See “Specifications” for sensor accuracy information.

Pulse Rate Display

The pulse rate display is located on the right-hand side of the Model 7500 front panel and is identified by the (♥) symbol. This display shows the pulse rate in beats per minute, from 18 to 321. The numeric displays blink during pulse rate alarm conditions. See “Specifications” for sensor accuracy information.

NOTE: LED means “light-emitting diode.”

888 Numeric LEDs

Green numeric LEDs display %SpO₂ and pulse rate values. When setting the device, these LEDs also display values for alarm limits, volume, and date and time settings.

Indicators and Icons



Alarm Bar LED

This LED indicates all alarm conditions. For high priority (patient) alarms, the indicator is displayed in red, blinking fast. For medium priority alarms, the indicator is displayed in amber, blinking slowly.



Pulse Quality LED

This amber LED blinks to indicate a poor pulse signal. If there is a sustained period of poor quality signals, this LED will display a steady, constant light.



Sensor Alarm LED

This amber LED indicates when a sensor has become disconnected, has failed, or is not compatible with this monitor.

WARNING: Do not use a damaged sensor. If the sensor is damaged in any way, discontinue use immediately and replace the sensor.



Pulse Strength Bargraph LED

This 8-segment tricolor bargraph indicates pulse strength as determined by the oximeter. The height of the Pulse Strength Bargraph LED is proportional to the pulse signal, and the color is determined by pulse strength:

Green = a good pulse strength

Amber = a marginal pulse strength

Red = a low pulse strength, high priority alarm



Alarm Silence LED

This amber LED indicates that the audible alarm is silenced for two minutes when it blinks. When alarms are active, this LED blinks in time with the alarm bar. If no alarms are active, this LED flashes at the medium priority alarm rate. When lit solid, the Alarm Silence LED indicates that audible alarm volumes are set to less than 45 dB.



AC Power Supply LED

This green LED is displayed when an external power supply is providing power to the Model 7500.

NOTE: When the external power supply is disconnected, the device automatically switches to battery power without loss of functionality.



Low Battery LED

This amber LED indicates a low battery charge when blinking, and a critical battery charge when lit solidly. *This LED does not indicate that the Model 7500 is running on battery power.*

WARNING: This device turns off after approximately 30 minutes when in low battery mode.

Model 7500 Front Panel Buttons



ON/STANDBY Button

Pressing this button once turns on the Model 7500. Holding this button for at least 1 second shuts down the 7500, putting it into Standby mode. In Standby mode, all device functions are shut off, with the following exceptions:

- The AC Power Supply LED is lit whenever the device is plugged in.
- Batteries are charged whenever the device is plugged in.

Momentarily pressing this button while the unit is on initiates an event marker.



Alarm Silence Button

This button toggles alarms between silenced and audible. Pressing the Alarm Silence button silences the alarm for two minutes. Pressing it again (while alarms are silenced) returns the alarms to their audible mode.



CAUTION: The two-minute alarm silence is automatically engaged at startup.



Limits Button

This button displays the upper and lower limits for alarm indications for SpO₂ and heart rate measurements.

Pressing the Limits button allows users to access advanced menu options, including adjusting alarm settings, alarm volume, oximetry averaging, and date and time settings. All adjustments can be made using the Plus (+) and Minus (-) buttons.



Plus (+) and Minus (-) Buttons

These buttons adjust values for many Model 7500 functions. The Plus (+) and Minus (-) buttons are used to adjust time, date, alarm volume, oximetry averaging, and upper and lower alarm limits, (except in Patient Security mode).



Operating the Model 7500

NOTES:

- Before using the Model 7500, please review all contraindications, warnings and cautions.
 - Before using the Model 7500, the battery must be charged for four (4) hours.
 - When the Model 7500 reaches critical battery, a medium priority alarm will sound. To clear the alarm: charge the battery and turn the device off and back on.
-

Press the ON/STANDBY button. When the unit is first turned on, the Model 7500 performs a brief initialization sequence.

Verify that all LEDs illuminate and the unit beeps three times during the first phase of the initialization sequence. If any LED is not lit (except the AC Power Supply LED), do not use the Model 7500. Contact Nonin Technical Service for assistance.

To verify that the Model 7500 is functioning properly, it is important to monitor SpO₂ and pulse rate readings. Use the following procedure to verify that the sensor is working properly.

1. Ensure that the Model 7500 is on, with the sensor connected.
2. Apply the pulse oximeter sensor (see sensor instructions for use).
3. Verify that a good SpO₂ reading is displayed, that a pulse rate value appears, and that the pulse strength bargraph LED is active.

WARNING: This device is intended only as an adjunct device in patient assessment. It must be used in conjunction with other methods of assessing clinical signs and symptoms.

WARNING: As with all medical equipment, carefully route patient cables and connections to reduce the possibility of entanglement, strangulation, or injury to the patient.

WARNING: Verify all device settings, alarm settings and limits in setup mode to ensure that they are set as intended.

Operating Modes and Defaults

The Model 7500 features Setup mode, Factory Defaults, User-Defined Defaults and Patient Security modes.

NOTE: Patient Security mode overrides any default settings.

Setup Mode, Viewing Limits and Setting Time

In Setup mode, users can adjust alarm limits and volumes, oximetry averaging, set clock and calendar information and clear the device's memory. Pressing the Limits button activates Setup mode, and all adjustments can be made using the Plus (+) or Minus (-) buttons. Setup mode is available when the device is operating, or during the startup/initialization process. Time is set by setting each of the last five options in setup mode: year, month, day, hour and minute.

Setup mode is not available in Patient Security mode. In Patient Security mode, pressing the Limits button scrolls through the limits on the displays, allowing the operator to view the current limits. Pressing and holding the Plus (+) button also reviews the limits, regardless of operating mode.

Factory Defaults

In Factory Defaults, all adjustable parameters are set as indicated in the table below. This is the Model 7500's default operating setting.

The Model 7500 is shipped with factory defaults active. To revert to Factory Default alarm limits from the User-Defined Default alarm limits, simultaneously press the alarm silence and minus (-) buttons.

NOTE: User-Defined Default values are lost when Factory Defaults are set active.

Table 2: Factory Default Settings

Parameter	Factory Default	Adjustment Options	Increment
SpO ₂ High Alarm Limit	Off	Off, 80-100	1%
SpO ₂ Low Alarm Limit	85%	Off, 50-95	1%
Pulse Rate High Alarm Limit	200 BPM	Off, 75-275	5 BPM
Pulse Rate Low Alarm Limit	50 BPM	Off, 30-110	5 BPM
Alarm Volume	High	Off, Low, High	N/A
Oximetry Averaging	Standard/Fast Averaged SpO ₂ (no)	Standard/Fast Averaged SpO ₂ (no), Hybrid Averaged SpO ₂ (yES)	N/A

Default alarm and volume settings are automatically selected for every operating session in which the parameters were not recalled or changed within the setup menu.

User-Defined Defaults

In User-Defined Defaults, alarm limit and volume settings may be adjusted. To set the User-Defined Defaults, set the alarm limits, hold the Alarm Silence button and then press the Limits button. This sets the User-Defined Defaults to be the same as the current alarm limits.

The Model 7500 recalls User-Defined Default settings at startup whenever this option is selected. Once activated, User-Defined Defaults have priority over Factory Defaults.

NOTE: All User-Defined Default settings are retained even when both external and battery power are lost.

Patient Security Mode

Alarm limits cannot be changed when the Model 7500 is in Patient Security mode. Patient Security mode prevents accidental changes to critical parameters. The Model 7500 allows users to lock and unlock alarm limits, oximetry averaging, volume settings, and time settings through the use of Patient Security mode. Operators will notice several operating differences with Patient Security mode:

- Default and other previous device settings cannot be recalled.
- Clock and calendar data cannot be changed.
- SpO₂ and pulse rate alarm limits and volumes cannot be changed. Pressing the Limits button allows the operator to review the limits.
- Oximetry averaging cannot be changed.
- Patient memory cannot be cleared.
- To put the device into Standby mode, the ON/STANDBY button must be held for at least 3 seconds.
- Memory playback not available.

Patient Security mode remains active even when the device is turned off and then turned back on. Patient Security mode is retained even when both external and battery power are lost.

NOTE: Turn on the device and verify Patient Security mode and settings after initiating Patient Security mode.

When the Patient Security mode is enabled, operators cannot change SpO₂, or Pulse Rate limits or Alarm Volume—though it is still possible to view those settings. In Patient Security mode, operators cannot view or set the time and date.

When the Model 7500 is turned on in Patient Security mode, “SEC on” is displayed in the display area, and three informational tones sound. The upper alarm limits are then displayed, followed by the lower alarm limits.

NOTE: Patient memory cannot be cleared when the Model 7500 is in Patient Security mode. In addition, Patient Security mode is not disabled when the unit is turned off.

Viewing and Changing Patient Security Mode

Enter Patient Security Mode – With the device off, press and hold the Alarm Silence button while turning the device on.

Exit Patient Security Mode – With the device off, press and hold the Alarm Silence and Limits buttons while turning the device on.








When the device is restarted, the Patient Security mode status is displayed on the Numeric LEDs for 1 second:

- “SEC on” is displayed when Patient Security mode is enabled.
- “SEC OFF” is displayed when Patient Security mode is disabled.

Operator Functions

The Model 7500 has several easy-to-use basic functions. Most involve pressing only a single button.

Table 3: Basic Functions

Function	Button	Instruction
Turn the Model 7500 on and off.		Press the ON/STANDBY button to turn on the Model 7500. Press and hold the button for at least one second to turn off the Model 7500. In Patient Security mode, hold the ON/STANDBY button for three seconds to turn off the Model 7500.
Initiate an event marker.		Momentarily press the ON/STANDBY button while the unit is on.
Mute the audible alarms (2 minutes).		Momentarily press the Alarm Silence button.
Change Pulse tone volume.		Momentarily press the Plus (+) button while the unit is in operating mode. Press again to sequence through volume options for pulse tones.
Set alarm limits or alarm volumes, clear memory or set clock.	 then  or 	Momentarily press the Limits button to step through the Limits menu. Use the Plus or Minus buttons to adjust alarm limits or selected volumes as desired. When pressing Limits button, settings will appear in the order shown in Table 4.



CAUTION: Review all limits to ensure they are appropriate for the patient.



CAUTION: Setting alarm limits to extremes can render the alarm system useless.

Table 4: Limits Display Sequence

Parameter	Parameter (SpO ₂) Display	Initial Setting (Pulse Rate Display)	Adjustment Range
Recall Alarm Settings	"rCL"	"no"	"yES" or "no"
Low %SpO ₂ Alarm Limit	"02L" ^{2,3}	"85"	"OFF", 50 to 95 by 1
Pulse High Alarm Limit	"HH" ²	"200"	"OFF", 75 to 275 by 5
Pulse Low Alarm Limit	"HL" ²	"50"	"OFF", 30 to 110 by 5
High %SpO ₂ Alarm Limit	"02H" ²	"OFF"	"OFF", 80 to 100 by 1

Table 4: Limits Display Sequence (Continued)

Parameter	Parameter (SpO ₂) Display	Initial Setting (Pulse Rate Display)	Adjustment Range
Alarm Volume	“adb” ²	“Hi”	“OFF” or “Lo” or “Hi”
Oximetry Averaging	“Hyb” ⁴	“no”	“yES” or “no”
Clear Memory	“CLr” ¹	“no”	“yES” or “no”
Confirm Memory Clear	“dEL” ¹	“no”	“yES” or “no”
Year	“y”	“00”	0 to 99 by 1
Month	“nn”	“00”	0 to 12 by 1
Day	“d”	“00”	1 to 31 by 1
Hour	“h”	“00”	0 to 23 by 1
Minutes	“nn”	“00”	0 to 59 by 1
<p>Notes:</p> <p>¹Both of these menu options are part of the memory clear command; “dEL” will be displayed only if “yES” was selected as the setting for the “CLr” parameter.</p> <p>²These parameters are restored when Recall Alarm Settings is set to “yES.” these are also the settings displayed by Review Alarm Settings.</p> <p>³The low SpO₂ Alarm limit saved for recall cannot be lower than the current default for that alarm limit. If it is, the default value will be used when alarm limits are restored.</p> <p>⁴The oximetry averaging restored is based off of previous selection.</p>			

The Model 7500 features a number of advanced options, which are intentionally more difficult to activate. These functions are recommended only for trained operators and they require multiple button presses to prevent accidental activation.

Table 5: Advanced Options






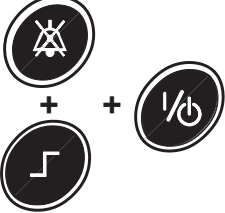


Function	Button	Instruction
Recall Previous Alarm Limit Settings	 	Press the Limits button while the unit is on. “rCL” appears, indicating that previous alarm limit settings may be recalled. To recall the settings, press the Plus button and select “yES.” Press the Limits button again to confirm.
Memory Playback	 	Press and hold the Plus (+) button while turning on the Model 7500. This functions with the Nonin nVISION [®] software. Select the Model 7500 option in nVISION software.

Table 5: Advanced Options (Continued)

Function	Button	Instruction
<p>NOTE: Alarm limits cannot be changed when the Model 7500 is in Patient Security mode. Patient Security mode prevents accidental changes to critical parameters. The Model 7500 allows users to lock and unlock alarm limits, volume settings, and time settings</p>		
Enter Patient Security Mode		To enter Patient Security mode, press and hold the Alarm Silence button while turning on the device.
Exit Patient Security Mode		To exit Patient Security mode, press and hold the Alarm Silence and Limits buttons while turning on the device.
Make Current Alarm Values User-defined Defaults		To set the User-Defined Defaults to the current alarm settings, hold the Alarm Silence button and then press the Limits button.
Revert to Factory Defaults		To revert to the factory defaults, from the User-Defined Defaults alarm limits, hold the alarm Silence button and then press the Minus (-) button.
<p>NOTE: User-defined default values will be lost when factory defaults are made active.</p>		

Care and Maintenance



CAUTION: Review all limits to ensure they are appropriate for the patient.

The advanced digital circuitry within the pulse oximeter of the Model 7500 requires no calibration or periodic maintenance other than battery replacement by Nonin Technical Service. The device's expected service life is 5 years.

Field repair of the Model 7500 circuitry is not possible. Do not attempt to open the Model 7500 case or repair the electronics. Opening the case may damage the Model 7500 and void the warranty. If the Model 7500 is not functioning properly, see "Troubleshooting."

The Oxitest^{Plus7} by Datrend Systems, Inc. can be used to verify operation of the pulse oximeter.



CAUTION: Follow local, state and national governing ordinances and recycling instructions regarding disposal or recycling of the device and device components, including batteries. Use only Nonin-approved battery packs. Batteries might leak or explode if used or disposed of improperly.

Cleaning the Model 7500

1. Clean the Model 7500 with a soft cloth dampened with isopropyl alcohol, mild detergent, or a 10% bleach (5.25% sodium hypochlorite) with water solution. Do not pour or spray any liquids onto the Model 7500, and do not allow any liquid to enter any openings in the device.
2. Allow the unit to dry thoroughly before reusing it.

WARNING: Do not use this device in or around water or any other liquid, with or without AC power.



CAUTION: Do not immerse this device in liquid, and do not use caustic or abrasive cleaning agents on the device. Do not gas sterilize or autoclave this device. Do not place liquids on top of this device.

Clean the Model 7500 separately from its associated sensors. For instructions regarding cleaning pulse oximeter sensors, refer to the appropriate pulse oximeter sensor package inserts.

Alarms and Limits

The Model 7500 is equipped with audio and visual alarm indicators to alert the operator to provide immediate patient attention or to abnormal device conditions.

The intended operator's position for correctly perceiving a visual alarm signal and its priority is 1 meter (3.3 feet), per IEC 60601-1-8.

High Priority Alarms

High priority alarms require immediate attention to the patient. They include SpO₂, pulse rate, and low perfusion alarms. On the Model 7500, high priority alarms are indicated by a rapidly blinking red Alarm Bar LED when the value is equal to or greater than the alarm limit. In addition, the pulse strength bargraph LED illuminates a red segment to indicate low perfusion.

High priority alarms are sounded as follows: three beeps, pause, two beeps and a 10 second pause.

Medium Priority Alarms

Medium priority alarms signal potential problems with the equipment or other non-life-threatening situations. On the Model 7500, medium priority alarms are indicated with a slowly blinking amber Alarm Bar LED.

Medium priority alarms are illuminated amber on the Alarm Bar LED and on the appropriate indicator(s) or numeric displays, sometimes displaying an error code to help the user identify the source of the error.

Medium priority alarms are sounded as three beeps and a 25-second pause.

Watchdog Alarms

Watchdog alarms are loud, two-tone, steadily beeping signals that indicate a hardware or software malfunction. When a watchdog alarm is activated, it can be cleared by shutting down the Model 7500. If the watchdog alarm cannot be cleared, remove power and contact your distributor or Nonin Technical Service.

Informational Tones

Informational tones communicate important information. They are typically single beeps or a series of three beeps. Informational tones include the startup/initialization tone and the pulse rate tone (which changes in pitch with SpO₂ values: higher tones for higher SpO₂, and lower tones for lower SpO₂).

Alarm Summary

The Model 7500 detects both patient and equipment alarms. In general, patient alarms are identified as high priority, while equipment alarms are identified as medium priority. High priority alarms always take priority over medium priority alarms. Alarm indicators remain active for as long as the alarm condition is present.

WARNING: Verify all alarm settings and limits during startup to ensure they are set as intended.

Patient Alarms

If patient SpO₂ or pulse readings are equal to or above the upper alarm limit, or if they are equal to or below the lower alarm limit, the device will signal a high priority alarm, indicated by numeric LEDs flashing in sync with the red Alarm Bar LED.

Table 6: Patient Alarms

Alarm Description	Factory Default	Adjustment Options	Increment
SpO ₂ High Alarm Limit	Off	Off, 80-100	1%
SpO ₂ Low Alarm Limit	85%	Off, 50-95	1%
Pulse Rate High Alarm Limit	200 BPM	Off, 75-275	5 BPM
Pulse Rate Low Alarm Limit	50 BPM	Off, 30-110	5 BPM
Low Perfusion Alarm	Red segment on Pulse Strength Bargraph indicates low patient perfusion.		

Equipment Alarms

Table 7: Equipment Alarms

Alarm Description	Visual Indicator
Low Battery Alarm	Battery LED blinks in sync with Alarm Bar LED. This alarm signifies that the battery has less than 30 minutes of normal operation. When Critical Low Battery is reached, the device's oximetry functions are disabled.
Sensor Alarm	Sensor Alarm LED blinks in sync with Alarm Bar LED. This alarm signifies a sensor alarm, disconnect, or fault.
Other Equipment Alarms	Error code appears in main display area.

Reviewing and Setting Volume and Alarm Limits

NOTE: Alarm limits reset themselves to default values each time the unit is powered up—unless the unit is in Patient Security mode. In Patient Security mode, alarm limits and volumes cannot be adjusted; they can only be viewed.

WARNING: To comply with relevant product safety standards, ensure that all alarm volumes are set appropriately and are audible in all situations. Do not cover or otherwise hinder any speaker openings.

Reviewing, Setting, or Changing Volumes and Alarm Limits

1. Ensure that the device is on.
2. Press the Limits button until the limit you want to view or change is displayed.
 - The current limit appears in the %SpO₂ display.
 - The current setting appears in the pulse rate display.
 - Continue to press the Limits button until the limit you want to change is displayed.
3. To change the displayed value, press the Plus (+) or Minus (-) buttons as desired.
4. Continue to press the Limits button until the unit returns to normal operation.



CAUTION: Review all limits to ensure they are appropriate for the patient.

Silencing Alarms

Press the Alarm Silence button to silence alarms for two minutes. The Alarm Silence LED blinks at the medium priority alarm rate while alarms are temporarily silenced. If alarms are silenced during active alarm conditions, the Alarm Silence LED blinks in time with the alarm bar.

The Alarm Silence LED will be lit solidly when the alarm volume is set to less than 45 dB. Audible indicators can be turned off in the Limits menu, by selecting "OFF" in the corresponding Alarm Volume menu option.

Recalling Previous Settings

The pulse oximeter includes a feature that allows recall of the operator-adjusted settings in use when the device was last turned off. The following settings are recalled when this feature is activated:

- SpO₂ high and low alarm limits
- Pulse rate high and low alarm limits
- Alarm volume settings
- Oximetry averaging settings

Previous operator-adjusted settings can be recalled by pressing the Limits button while the unit is on. “rCL” appears, indicating that previous alarm limit settings may be recalled. To recall the settings, press the Plus button and select “yES.” Press the Limits button again to accept the recall and return to normal operation.



CAUTION: Review all limits to ensure they are appropriate for the patient.

NOTE: The recalled value for the SpO₂ low alarm will not be less than the current default.

Error Codes

This device includes error codes that indicate problems with the unit. Error codes are indicated by “Err” in the %SpO₂ display, and a capital “E” followed by a 2-digit code in the pulse rate display. To correct error conditions, perform the following steps:

1. Turn the unit off and then back on again to remove the error code.
2. If the error persists, disconnect all power, and then reconnect the power and turn the unit back on.

If the error still persists, note the error code and contact Nonin Technical Service at (800) 356-8874 (USA and Canada), +1 (763) 553-9968, or +31 (0)13 - 45 87 130 (Europe).

Memory and Data Output Features

The Model 7500 provides real-time (serial) patient data output, as well as analog output signals for SpO₂, pulse rate, and event markers.

Serial Patient Data Output

This device features real-time data output capabilities. The serial format includes an ASCII header containing model number, time, and date information.

The device provides real-time data output capability via the serial connector port. The 7500 SC cable, available from Nonin, may be used to connect the Model 7500 to the receiving computer. The information from the Model 7500 is sent in an ASCII serial format at 9600 baud with 8 data bits, 1 start bit, and 2 stop bits. Each line is terminated by CR/LF.

Data from the device are sent once per second in the following format:

SPO₂=XXX HR=YYY

NOTE: Pressing the ON/STANDBY button will insert an "*" at the end of the corresponding printed line to serve as an event marker.

Table 8: Real-time Data Output

Pin Number	Pin Assignment
1	Analog Output, SpO ₂
2	No Connect
3	Serial Data Output
4	Analog Output, Pulse Rate
5	Ground
6	No Connect
7	Event Marker
8	No Connect
9	No Connect

Analog Output

The Model 7500 provides analog output signals for SpO₂, pulse rate, and event markers. Each output level conforms to the specifications shown below:

Output	Specification
SpO ₂ Output Analog Range	0 - 1.0 VDC (representing 0-100%) 1.27 VDC (out of track)
Pulse Rate Output Analog Range	0 - 1.0 VDC (representing 0-300 BPM) 1.27 VDC (out of track)
Event Marker	0 VDC or 1.0 VDC nominal (representing an event). Event marker high for SpO ₂ less than low alarm limit.
Analog Output Load Current	2 mA maximum
%SpO ₂ Analog Output Accuracy	±2%
Pulse Rate Analog Output Accuracy	±5%

Analog Output Calibration

Analog calibration signals that allow external device calibration are provided after initial power up, and continue until the Model 7500 begins tracking SpO₂ and pulse rate readings. The calibration routine ends when the system begins tracking signals. The calibration signal sequence is as follows:

Time Interval	Analog Signal
30 seconds	1.0 VDC
30 seconds	0.0 VDC
1 second	0.1 VDC
1 second	0.2 VDC
1 second	0.3 VDC
1 second	0.4 VDC
1 second	0.5 VDC
1 second	0.6 VDC
1 second	0.7 VDC
1 second	0.8 VDC
1 second	0.9 VDC
1 second	1.0 VDC
1 second	1.27 VDC
Repeat	

Memory Features

The Model 7500 can collect and store 70 hours of continuous SpO₂ and pulse rate information.

Data may be played back with data retrieval software (Nonin's nVISION software is recommended). If you wish to create your own software, contact Nonin for the data format.

The memory in the Model 7500 functions much like an "endless loop" tape. When the memory is full, the unit begins overwriting the oldest data with new data. Data is written in 4-minute intervals.



CAUTION: If the entire memory is filled, portions of the oldest record will be overwritten when a new record begins.

Each time the Model 7500 is turned on, the current time/date information (if the clock is set properly) is stored in memory, starting a new recording session. Only recording sessions greater than one minute in length are stored in memory.

Patient SpO₂ and pulse rate are sampled every second. Every 4 seconds, the extreme value of the 4-second sample period is stored. Oxygen saturation values are stored in 1% increments in the range of 0 to 100%.

The stored pulse rate ranges from 18 to 300 pulses per minute. The stored values are in increments of one pulse per minute in the interval from 18 to 200 pulses per minute, and in increments of 2 pulses per minute in the interval from 201 to 300 pulses per minute.

Patient data is retained even when both external and battery power are lost.

Clearing Patient Memory

Patient memory can be cleared using the Model 7500's Setup mode. Press the Limits button to enter Setup mode, and use the Limits button again to scroll through the device's options until Memory Clear is displayed. Select "Yes" or "No" using the Plus (+) or Minus (-) buttons to clear patient memory, and then confirm your selection using the Limits button.

Playing Back Memory Data

The Model 7500 has a Memory Playback feature, allowing stored data to be output through an external serial connection. Playing back the data does not clear the data from memory.

1. With the unit off, connect the serial connector port of the Model 7500 to the back of your computer using the 7500SC cable, which is available from Nonin.
2. Press and hold the Plus (+) button while briefly pressing the ON/STANDBY button.
3. Release the Plus (+) button. Playback mode will be shown on the SpO₂ and Pulse Rate displays until memory playback is completed.
4. When Memory Playback is complete, the device will return to normal operation.

NOTES:

- Patient memory cannot be cleared when the Model 7500 is in Patient Security mode.
 - If using nVISION software, select "Model 7500" for model type.
 - E07 is an error code which occurs during memory playback and signifies that the unit cannot guarantee the accuracy of all patient data in memory. Clear the device memory to ensure any inaccurate data is deleted. If the error continues to occur on memory playback, call Nonin Technical Service.
 - The event marker is not stored in the 7500 memory.
-

Connecting the Device into a Medical System

Incorporating the device into a medical system requires the integrator to identify, analyze, and evaluate the risks to patient, operators, and third parties. Subsequent changes to the medical system after device integration could introduce new risks and will require additional analysis. Changes to the medical system that must be evaluated include:

- Changing the system configuration
- Adding devices to or disconnecting devices from the system
- Updating or upgrading equipment connected to the system

Issues resulting from user-initiated system changes may include corruption or loss of data.

NOTES:

- Use of a multiple-socket outlet with multiple devices results in a Medical Electrical System.
- When using the serial port to connect the device to other equipment, follow each device's cleaning instructions.
- Verify all equipment connected to the device is suitable for the patient's environment.



CAUTION: Failure of a network data coupling (serial cable/connectors) will result in loss of data transfer.

Service, Support, and Warranty

A return authorization number is required before returning any product to Nonin. To obtain this return authorization number, contact Nonin Technical Service:

Nonin Medical, Inc.

13700 1st Avenue North
Plymouth, Minnesota 55441, USA

(800) 356-8874 (USA and Canada)
+1 (763) 553-9968 (outside USA & Canada)
Fax: +1 (763) 553-7807
E-mail: technicalservice@nonin.com

Nonin Medical B.V.

Doctor Paul Janssenweg 150
5026 RH Tilburg, Netherlands

+31 (0)13 - 45 87 130 (Europe)
E-mail: technicalserviceintl@nonin.com



CAUTION: This device is a precision electronic instrument and must be repaired by Nonin Technical Service. Field repair of the device is not possible. Do not attempt to open the case or repair the electronics. Opening the case may damage the device and void the warranty.

Warranty

For warranty information go to: <http://www.nonin.com/warranty/>

Parts and Accessories

For more information about Nonin parts and accessories:

- See the Parts and Accessories List on the Operator's Manual USB Drive.
- Contact your distributor or Nonin at (800) 356-8874 (USA and Canada), +1 (763) 553 9968, or +31 (0)13 - 45 87 130 (Europe).
- Visit www.nonin.com.

WARNING: Use this device only with Nonin-specified power supplies.

WARNING: The use of accessories, sensors, and cables other than those listed in the Parts and Accessories List may result in increased electromagnetic emission and/or decreased immunity of this device.

WARNING: To avoid patient injury, use only Nonin-branded PureLight pulse oximeter sensors. These sensors are manufactured to meet the accuracy specifications for Nonin pulse oximeters. Using other manufacturers' sensors can result in improper pulse oximeter performance.

Troubleshooting


Problem	Possible Cause	Possible Solution
Model 7500 will not activate.	The unit has no power.	Plug in the AC power supply.
Model 7500 will not operate on batteries.	The battery pack is not charged.	Plug in the Model 7500 AC power supply to charge the battery pack.
	The battery pack is inoperable.	Contact Nonin Technical Service for repair or replacement.
Unable to obtain a green pulse display on the bargraph. NOTE: In some instances, patient perfusion may be inadequate for pulse detection.	The patient pulse strength is indiscernible or perfused poorly.	Reposition the finger or insert a different finger, and keep the sensor motionless for at least 10 seconds. Warm the patient's finger by rubbing or covering with a blanket. Position the sensor at a different site.
	Circulation is reduced because of excess pressure on the sensor (between the sensor and a hard surface) after inserting finger.	Allow the hand to rest comfortably without squeezing or pressing the sensor on a hard surface.
	The finger is cold.	Warm the patient's finger by rubbing or covering with a blanket. Position the sensor at a different site.
	The sensor is applied incorrectly.	Apply the sensor correctly.
	There is possible interference from one of the following sources: <ul style="list-style-type: none"> • arterial catheter • blood pressure cuff • electrosurgical procedure • infusion line 	Reduce or eliminate any interference. Make sure that the sensor is not placed on the same arm being used for other patient therapies or diagnostics (e.g., blood pressure cuff).
	The red LED is not lit in the sensor's finger insertion area.	Ensure the sensor is securely attached to the Model 7500.
		Check the sensor for any visible signs of deterioration.
		Contact Nonin Technical Service.


Problem	Possible Cause	Possible Solution
Frequent or steady pulse quality indication.	There is excessive ambient light.	Shield the sensor from the light source.
	The Model 7500 is applied to a polished or artificial fingernail.	Apply the sensor to a finger without artificial or polished nails.
		Position the sensor at a different site.
	The red LED is not lit in the sensor's finger insertion area.	Ensure the sensor is securely attached to the Model 7500.
		Check the sensor for any visible signs of deterioration.
		Contact Nonin Technical Service.
Excessive patient motion.	Reduce patient motion.	
A dash (-) appears in the %SpO₂ display.	An inadequate signal from the finger is being detected.	Reposition finger or insert a different finger, keeping sensor motionless for at least 10 seconds.
		Position sensor at different site.
	The finger was removed from the sensor.	Reinsert the finger and keep the sensor motionless for at least 10 seconds.
	The Model 7500 is not functioning.	Turn the unit off, check all connections, and retry.
		Contact Nonin Technical Service.
An error code appears in the display area.	The Model 7500 encountered an error.	Turn the unit off and then back on again to remove the error code. If the error persists, disconnect all power, and then reconnect the power and turn the unit back on. If the error still persists, note the error code and contact Nonin Technical Service.
The unit is in Alarm mode, but no audible alarms can be heard.	The 2-minute Alarm Silence button is activated.	Press the Alarm Silence button to re-engage alarm volume, or wait for 2 minutes. After 2 minutes, alarm tones will automatically re-engage.
	Audible volume set to "OFF" in alarm limits.	Adjust volume through setup mode
The Model 7500 does not record data.	The battery is low.	Recharge the battery.
	The battery is missing.	Contact your distributor or Nonin Technical Service for repair or replacement.

If these solutions do not correct the problem, please contact Nonin Technical Service at (800) 356-8874 (USA and Canada), +1 (763) 553-9968, or +31 (0)13 - 45 87 130 (Europe).

Technical Information

NOTE: This product complies with ISO 10993, Biological Evaluation of Medical Devices Part 1: Evaluation and Testing.

 **CAUTION:** A functional tester cannot be used to assess the accuracy of a pulse oximeter monitor or sensor.

 **CAUTION:** All parts and accessories connected to the serial port of this device must be certified according to at least IEC Standard EN 60950, IEC 62368-1, or UL 1950 for data-processing equipment.

WARNING: Portable RF communications equipment such as cell phones or radios (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the ME system, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

Manufacturer’s Declaration

Essential Performance

Essential Performance of the Model 7500 Pulse Oximeter includes SpO2 Accuracy, Pulse Rate Accuracy and Limit Alarm Conditions or generation of a Technical Alarm Condition. Accuracies or alarms may be affected as a result of exposure to electromagnetic disturbances that are outside of the environments listed in the Indications For Use. If issues are experienced, move Nonin system away from the source of electromagnetic disturbances.

Refer to the following table for specific information regarding this device’s compliance to IEC 60601-1-2.

Table 9. Electromagnetic Emissions

Emissions Test	Compliance
<i>This device is intended for use in the electromagnetic environment specified in the Indications For Use.</i>	
RF Emissions CISPR 11	Group 1, Class B
Harmonic Emissions IEC 61000-3-2	Class A
Voltage Fluctuations/Flicker Emissions IEC 61000-3-3	Within limits of IEC 61000-3-3

Table 10. Electromagnetic Immunity

Immunity Test	Compliance Level	
<i>This device is intended for use in the electromagnetic environment specified in the Indications For Use.</i>		
Electrostatic Discharge (ESD) IEC 61000-4-2	±8 kV contact ±15 kV air	
Electrical Fast Transient/Burst IEC 61000-4-4	±2 kV for power supply lines ±2 kV for input/output lines	
Surge IEC 61000-4-5	±0.5kV, ±1kV, ±2kV for Line-to-Ground ±0.5kV, ±1kV for Line-to-Line	
Voltage dips, short interruptions, and voltage variations on power supply input lines IEC 61000-4-11	0% UT for 0.5 Cycles at 50 Hz at 0, 45, 90, 135, 180, 225, and 315° Phase 0% UT for 1 Cycle at 50 at 0°Phase 70% UT for 25 Cycles at 50 Hz at 0°Phase 0% UT for 250 cycles at 50 Hz at 0°Phase	
Power Frequency (50/60 Hz) Magnetic Field IEC 61000-4-8	30 A/m	
Conducted RF IEC 61000-4-6	150 kHz to 80 MHz	3 Vrms
	ISM and Amateur radio bands between 150 kHz to 80 MHz	6 Vrms
Radiated RF IEC 61000-4-3	80 MHz – 2.7 GHz	10 V/m
	380 – 390 MHz	27 V/m
	430 – 470 MHz	28 V/m
	704 – 787 MHz	9 V/m
	800 – 960 MHz	28 V/m
	1.7 – 1.99 GHz	28 V/m
	2.4 – 2.57 GHz	28 V/m
5.1 – 5.8 GHz	9 V/m	
Note: U_T is the AC mains voltage before application of the test level.		

Oximetry Averaging for Displayed SpO₂

The SpO₂ value displayed on the monitor screen is the average of the data collected within a specific time. The 7500 allows for selection between two oximetry averaging, Standard/Fast or Hybrid averaging.

Standard/Fast averaging is the default averaging set on the device. Standard/Fast averaging responds to changes in the patient’s oxygen saturation level faster than Hybrid averaging. This can more quickly identify transitory and sudden desaturation events.

Hybrid Averaging has a slower equipment response to patient’s oxygen desaturation to minimize any sudden SpO₂ value changes due to motion artifact or other transient conditions, while still providing the rapid recovery of Standard/Fast averaging.

Equipment Response Time

If the signal from the sensor is inadequate, the last measured SpO₂ and pulse rate values freeze for 10 seconds and are then replaced with dashes.

Selecting Standard/Fast Averaging utilizes 4 beat exponential averaging for both SpO₂ and Pulse Rate.

Selecting Hybrid Averaging utilizes 4 - 8+ beat adaptive averaging for the SpO₂ values, and 8 beat exponential averaging for the Pulse Rate.

Oximetry Averaging	SpO ₂ Values		Pulse Rate Values	Latency
Standard/Fast	Desaturation	4 beat exponential	4 beat exponential	2 beats
	Saturation	4 beat exponential	4 beat exponential	2 beats
Hybrid	Desaturation	8+ beat exponential*	8 beat exponential	2 beats
	Saturation	4 beat exponential	8 beat exponential	2 beats


*The Hybrid Desaturation SpO₂ exponential is a Nonin derivation of 8 beat exponential.

Equipment Delays	Delay
Display Update Delay	1.5 seconds
Alarm Signal Generation Delay	0 seconds

Example - SpO₂ Exponential Averaging

SpO₂ decreases 0.75% per second (7.5% over 10 seconds)

Pulse Rate = 75 BPM

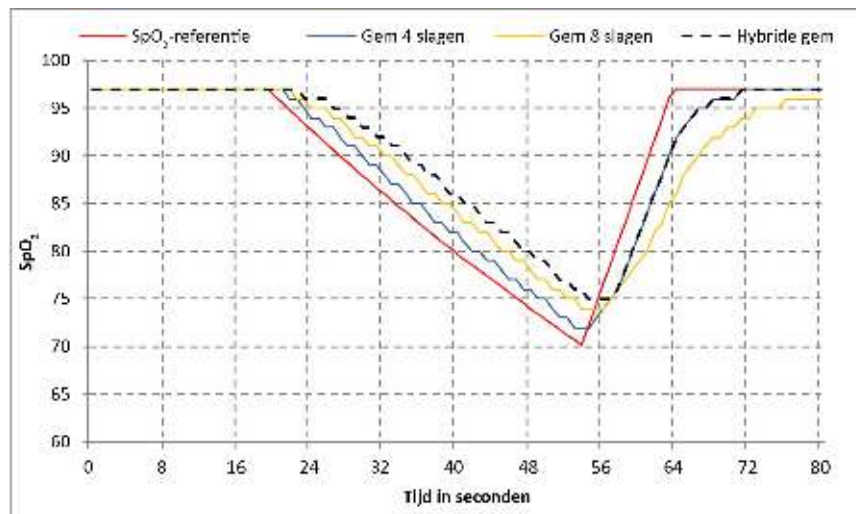

Specific to this example:

- Response of the 4-beat averaging -
 - Desaturation response is 1.5 Seconds.
 - Recovery response is 1.5 Seconds.
- Response of the Hybrid averaging -
 - Desaturation response is 5 Seconds.
 - Recovery response is 1.5 Seconds.

Testing Summary

SpO₂ accuracy, motion, and low perfusion testing were conducted by Nonin Medical, Inc., as described below:

SpO₂ Accuracy Testing

During motion and no-motion conditions at an independent research laboratory, SpO₂ accuracy testing is conducted during induced hypoxia studies on healthy, male and female, non-smoking, light- to dark-skinned subjects that are 18 years of age and older. The measured arterial hemoglobin saturation value (SpO₂) of the sensors is compared to arterial hemoglobin oxygen (SaO₂) value, determined from blood samples with a laboratory co-oximeter. The accuracy of the sensors in comparison to the co-oximeter samples measured over the SpO₂ range of 70 –100%. Accuracy data is calculated using the root-mean-squared (A_{rms} value) for all subjects, per ISO 80601-2-61, Medical Electrical Equipment—Particular requirements for the basic safety and essential performance of pulse oximeter equipment for medical use.

Pulse Rate Motion Testing

This test measures pulse rate oximeter accuracy with motion artifact simulation introduced by a pulse oximeter tester. This test determines whether the oximeter meets the criteria of ISO 80601-2-61 for pulse rate during simulated movement, tremor, and spike motions.

Low Perfusion Testing

This test uses an SpO₂ Simulator to provide a simulated pulse rate, with adjustable amplitude settings at various SpO₂ levels for the oximeter to read. The oximeter must maintain accuracy in accordance with ISO 80601-2-61 for heart rate and SpO₂ at the lowest obtainable pulse amplitude (0.3% modulation).

Principles of Operation

Pulse oximetry is a non-invasive method that passes red and infrared light through perfused tissue and detects the fluctuating signals caused by arterial pulses. Well-oxygenated blood is bright red, while poorly oxygenated blood is dark red. The pulse oximeter determines functional oxygen saturation of arterial hemoglobin (SpO₂) from this color difference by measuring the ratio of absorbed red and infrared light as volume fluctuates with each pulse.

Specifications

Oxygen Saturation Display Range:	0 to 100% SpO ₂
Pulse Rate Display Range:	18 to 321 pulses per minute (BPM)
Displays:	Pulse Quality: LED, amber Sensor Alarm: LED, amber Pulse Strength Bargraph: LED, bargraph, tri-color segments Alarm Indicator: LED, bi-color Alarm Silenced: LED, amber Numeric Displays: 3-digit, 7-segment LEDs, green Low Battery: LED, amber
Accuracy - Sensors:	Declared accuracy for compatible sensors can be found in Nonin's Sensor Accuracy document.
Alarm Volume:	High: 75 dBA Low: 63 dBA
Informational Tone Volume:	High: 67 dBA Low: 55 dBA
Measurement Wavelengths and Output Power:^a	Red: 660 nm @ 0.8 mW maximum average power Infrared: 910 nm @ 1.2 mW maximum average power
Memory:	70 hours (assuming continuous operation)
Temperature:	Operating: 0 °C to 40 °C (32 °F to 104 °F) Storage/Transportation: -40 °C to 70 °C (-40 °F to 158 °F) Time (from storage) for monitor to be ready for its intended use: 6 minutes to warm from -40 °C to 0 °C 23 minutes to cool from 70 °C to 40 °C
Humidity:	Operating: 10 to 90% noncondensing Storage/Transportation: 10 to 95% noncondensing
Altitude (Operating):	Up to 4,000 meters (13,123 feet)
Hyperbaric Pressure:	Up to 4 atmospheres

a. This information is especially useful for clinicians performing photodynamic therapy.

Power Requirements (Mains):	100-240 VAC 50-60 Hz
------------------------------------	----------------------

Internal Power:	
	Battery: 7.2 volt NiMH battery pack
	Operating Life (fully charged battery): 16 hours minimum
	Storage Life: 21 days minimum
	Recharge Rate: 4 hours maximum
Dimensions:	Approximately 219 mm (8.6") W x 92 mm (3.6") H x 142 mm (5.6") D
Weight:	Approximately 900 grams (2 lbs) with battery
Warranty:	3 years
Classification per ANSI/AAMI ES60601-1 and CAN/CSA-C22.2 No. 60601-1:	
	Type of Protection: Class II (on AC power with MPP30 power supply). Internally powered (on battery power)
	Degree of Protection: Type BF-Applied Part
	Mode of Operation: Continuous
	Enclosure Degree of Ingress Protection: IP33
Analog Outputs:	
	SpO ₂ Output Range: 0-1 VDC (0-100% SpO ₂), 1.27 VDC (out of track)
	Pulse Rate Output Range: 0-1 VDC (0-300 BPM), 1.27 VDC (out of track)
	Event Maker: 0 V (no event), 1 V (event occurred)
	Accuracy: ±2% (SpO ₂), ±5% (Pulse Rate)
	Load Current: 2 mA maximum